

DARAY®

DARAY®

For returns policy and
warranty information
please visit:

www.daray.co.uk

Tel: 0844 375 9000 fax: 0333 321 0973
email: info@daray.co.uk url: www.daray.co.uk
Marquis Drive, Moira, Derbyshire, DE12 6EJ

QAM-HL550LED-1-0811



HL550 HEADLIGHT OPERATING MANUAL

1. Introduction


The **DARAY** HL550 head-light is lightweight and comfortable to use.

With the advantages of LED technology, the HL550 has no heat in the beam and is extremely efficient (can be used for up to 100,000 hours), meaning it is a cost effective head-light.


The LED head-light emits a near-daylight coloured beam ensuring tissue tones are accurately rendered.

2. Features

- 3W LED
- Focusable beam
- Tilting head
- Rechargeable

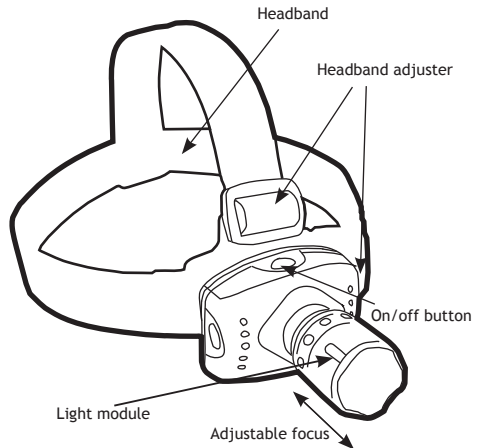

Connecting adapter

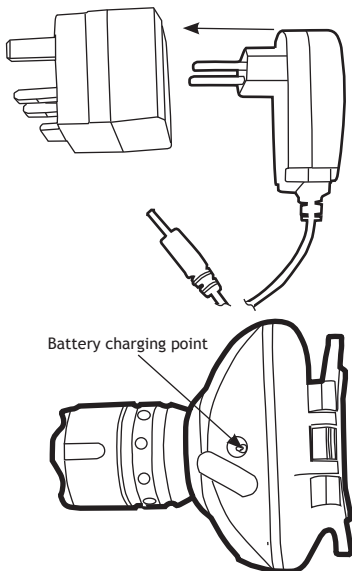

3. Operation

- Press the green button once for head-light to turn on
- Once the head-light is turned on, press the green button again for a dimmer light.
- Once the head-light is on the dim setting press the green button again for the light to flash on and off.
- Once the light is in flashing mode press the green button again to turn off the head-light.
- To adjust focus, move the light module forward and backward.